
Dainamed®

Scars

SILICONE SHEET
PLANCHA DE SILICONA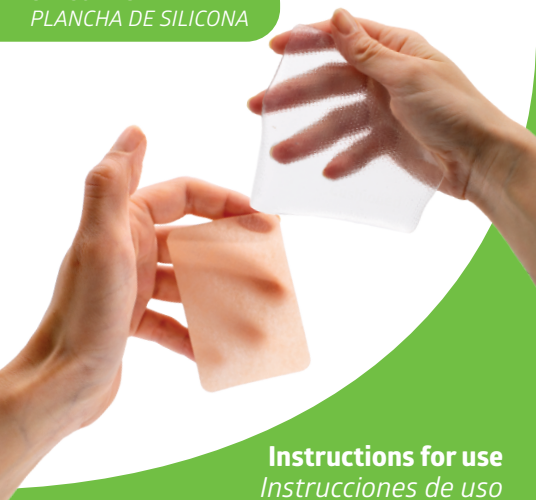

Instructions for use Instrucciones de uso

EN Silicone gel sheet for scar care / **ES** Plancha de silicona para el cuidado de cicatrices / **FR** Feuille de gel de silicone pour les soins des cicatrices / **DE** Silikongel Blatt zur narbenpflege / **IT** Gel in silicone per il trattamento delle cicatrici / **PT** Lâmina de gel de silicone para o tratamento de cicatrizes / **RU** Лист силиконового геля для ухода за шрамами / **SV** Silikonark, behandling av ärr och brännskador / **FI** Silikonisiteet. Arpien ja palovammojen hoito / **AR** لاصقة سيليكون جل للندبات / **VN** Hướng dẫn sử dụng miếng dán sẹo dainamed / **עברית** פלטת סיליקון לטיפול בצלקות / **CZ** Silikonové náplasti pro péči o jizvy / **SK** Silikónové náplasti na ošetrovanie jaziev

Reference	Size	Units	Shape	Type
F-02565-01CDA	5 x 7,5 cm	5		Skin feel
F-02565-14CDA	4 x 30 cm	5		Skin feel
F-02565-15CDA	10 x 18 cm	5		Skin feel
F-02555-01CDA	5 x 7,5 cm	5		Cushioned
F-02555-02CDA	10 x 10 cm	1		Cushioned
F-02555-03CDA	30 x 40 cm	1		Cushioned
F-02555-04CDA	40 x 50 cm	1		Cushioned
F-02555-13CDA	12 x 15 cm	10		Cushioned
F-02555-14CDA	12 x 15 cm	1		Cushioned
F-02555-06CDA	5 x 20 cm	2		Cushioned
F-02555-55CDA	10 x 30 cm	Pair		Cushioned
F-02555-56CDA	7 x 16 cm	2		Cushioned

EN Measure	Cut	Remove plastic	Apply
ES Medir	Cortar	Retirar plástico	Aplicar
FR Mesure	Coupez	Retirez le plastique	Appliquez
PT Medir	Cortar	Retirar o plástico	Aplicar
IT Misurare	Tagliare	Rimuovere la plastica	Applicare
DE Mät	Klipp	Ta bort plast	Applicera
AR القياس	قص	إزالة البلاستيك	تطبيق
RU Измерьте	Отрежьте	Снимите пластиковую пленку	Наложите
SV Mät	Beskär	Ta bort plastfilmen	Applicera
FI Mittaa	Leikkaa	Poista muovikelmu	Kiinnitä
VN Đo lường	Cắt	Loại bỏ nhựa	Áp dụng
CZ Poměřte	Zastříhněte	Odstraňte fólii	Aplikujte
SK Zmerajte	Upravte	Odstráňte fóliu	Aplikujte

Click here for **English**
 Clic aquí para **Español**
 Cliquez ici pour **Français**
 Clique aqui para **Português**
 Clicca qui per **l'italiano**
 Klicken Sie hier für **Deutsch**
 انقر هنا للغة العربية
 Нажмите здесь для **русского** языка
 Klicka här för **Svenska**
 Klikkaa tästä **suomen kieleen**
 Bấm vào đây để đọc tiếng **Việt**
 תירבעל נאכ **עברית**
 Klikněte pro **Češtinu**
 Kliknite pre **Slovenčinu**

Silicone sheets

Scars & burns treatment

SKIN FEEL	CUSHIONED
All types of scars and burns, including keloids and hypertrophic scars	All types of scars and burns, including keloids and hypertrophic scars
UPF 15 sun protection	-
0,4mm thickness (+/-1%)	1,7mm thickness (+/-3%)
Semi-transparent beige: with skin feel	Transparent: with a smooth and soft feel
Lighter and thinner, the best option for everyday use	Extra cushioning, best option to combine with bandages and pressure garments

1. Description

- Fresco's Dainamed silicone gel adhesive sheets are a medical device that helps to improve the healing of all types of scars and burns, whether recent or old.
- Treat and prevent keloids and hypertrophic scars.
- Fade and flatten all types of scars and reduce redness, pain and itching.
- Washable and reusable. Can be reused up to 20 times, depending on how the adhesive part is cared for.
- The ideal treatment should last 3 to 4 months. It may even last 6 months in the case of a more severe scar.

2. Who they are for

Anyone with scars and burns on any part of the body.
Designed for use in homes and healthcare facilities.

3. How they work

- By occlusion, reducing transepidermal water loss and keeping the skin hydrated.
- Reduce tension in the affected area, limiting the movement of the skin. Dainamed silicone sheets contain an atraumatic adhesive, which does not damage the skin when the sheet is removed.

In the case of Dainamed Scars Cushioned, the greater thickness has more padding, which makes them perfect for combining with bandages and compression garments.

4. Instructions for use

1. Begin treatment once the wound is completely closed or 3 days after removal of stitches.

2. Clean and dry the affected area

3. Dainamed Scars Skin feel: remove the transparent plastic protective film. Keep the film together with the packaging for when the product is not in use.

Dainamed Scars Cushioned: remove the two protective plastic films, keep the white film together with the packaging for when the product is not in use, and the transparent one can be discarded.

4. The scar must be completely covered. Leave at least 7 mm distance between the scar and the edge of the silicone sheet. If necessary, fix the sheet in place with a bandage or adhesive tape.

5. To allow the skin to adapt to the silicone sheet:

- I. Start treatment by applying them for 2 hours a day for 4 days.
- II. Increase to 4 hours a day for the next two days.
- III. Increase the application time by 2 hours each day until the optimum treatment is reached, which is 24 hours per day.
- IV. The duration of treatment should be monitored by a physician, but a minimum of one month of use is recommended.

5. Cleaning

Designed to be washable and reusable for the same patient.

Wash with warm water and pH-neutral soap, then leave to air dry before reuse.

Disposal at the end of useful life: Must be replaced when: Shows visible wear, is difficult to clean and/or has lost adhesion. At the end of its useful life, clean the sheet and dispose of it with non-hazardous waste.

6. Warnings and precautions

Do not use on open or oozing wounds. For external use only. For use on a single patient. Inspect the skin periodically.

Discontinue use of the product and seek advice from a healthcare professional if:

- The skin loses its integrity, i.e., it deteriorates.
- There is a deep tissue lesion or one is suspected.
- There is skin irritation.
- Heat rashes or reddening of the skin occur.

Sweating may occur if used in hot environments. This may decrease the effectiveness of the adhesive. To prevent this problem, wash the sheet twice a day and adapt its use to the temperature conditions, using a retention bandage or hypoallergenic adhesive tape. In this case, do not compress the area of application excessively, as this may cause irritation.

As the silicone gel sheets prevent the skin from breathing normally, some redness may be observed on the area of application. If this happens, reduce treatment to 12 hours per day or less. If skin redness persists, if there is pain or irritation, discontinue treatment and consult your physician.

Do not expose the product or the scar to direct sunlight.

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**FRESCO INTERNATIONAL 2005, S.A.**